


RIDA®SMART APP Allergen – measurement workflow

1 Start – standard mode/serial mode

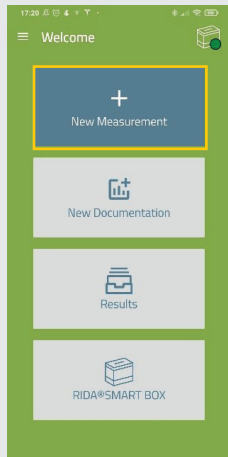


Start the RIDA®SMART APP Allergen and press „New Measurement“.

Follow point 2a for the workflow in the **standard mode**.

Follow point 2b for the workflow in the **serial mode**.

2a Measurement – standard mode

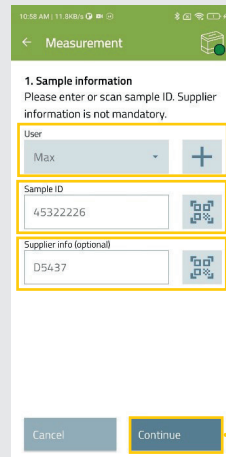


Select user.

Enter sample ID or scan the barcode.

Enter supplier info.


Tap „Continue“.

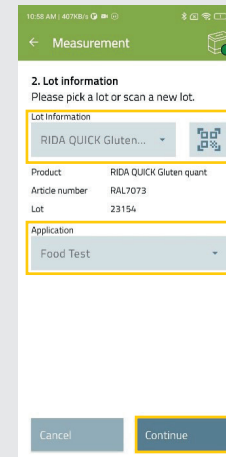

Choose „Lot Information“.

Choose „Application“.

Tap „Continue“.

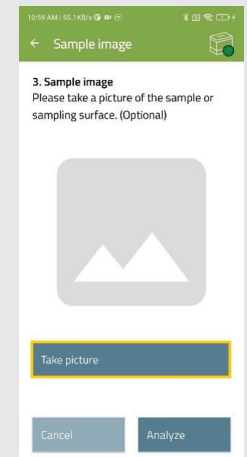

2b Measurement – serial mode



Select user.


Enter sample ID or scan the barcode.

Choose „Lot Information“.

Choose „Application“.

Tap „Analyze“.

3 Analysis – standard mode/serial mode

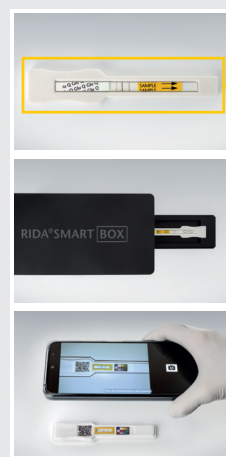

Strip holder


RIDA®SMART BOX:

To analyze, position the test strip in the strip holder and place the strip holder in the drawer of the RIDA®SMART BOX. Close the drawer.

RIDA®SMART APP:

To analyze, position the test strip in the strip holder and place the color cover above. Tap „Continue“.

4 Result – standard mode/serial mode

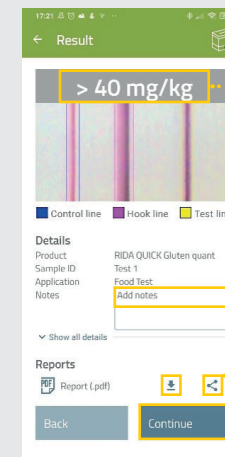

Find your result

Add notes to your result

Share or download your result

Tap „Continue“ to finalize your result