


# RIDA®SMART APP Mycotoxin

## 1 Start – registration


After installing the RIDA®SMART APP on your Android device, open the app and create an account. You can then log in and get started.

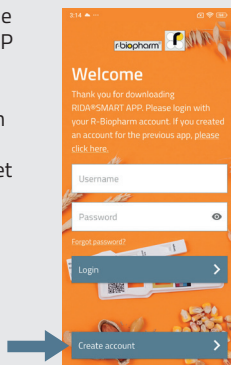

## 2 Measurement

To measure with your smartphone, tap "New Measurement" to start.


Using the **RIDA®SMART BOX?** First connect it to your Android device via "RIDA® SMART BOX" and follow the quick connection guide.

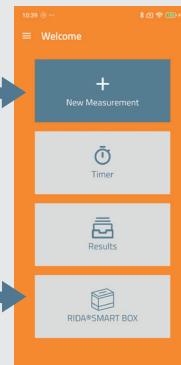


1. Define your sample ID. Supplier information is optional. Click "Continue".



2. Scan your Kit lot with QR code from your certificate of analysis by using the scan button. Choose your application. Click "Continue".

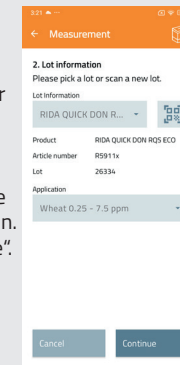


3. Take a picture of your sample (optional). Click "Continue" to start the measurement.

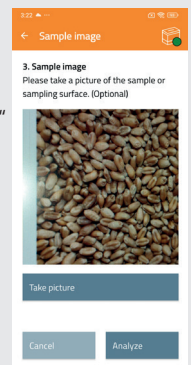

## 3 Analysis – RIDA®SMART BOX


**By RIDA®SMART BOX:**

Insert the test strip, close the drawer, and start the measurement. For measurement by RIDA®SMART BOX benchtop device.

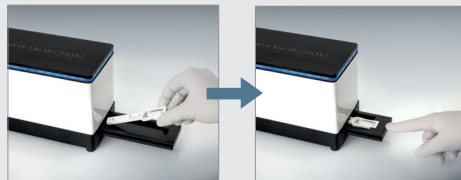

**By smartphone:**

Place the cover (from the kit) on the test strip. Align the phone and tap the camera button to capture.

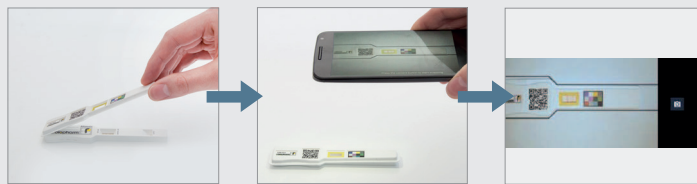

## 4 Result – data management

**Result & data export:**

After the measurement, the result screen appears automatically. The result is uploaded to RIDA®SMART Universe and can be exported as PDF, CSV, or to the cloud (see data export quick guide).



➔ RIDA®SMART UNIVERSE

